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Rad-X is a radiological imaging application used to diagnose and monitor the examinations performed. All normal/abnormal body, tissue, cell appearances and conditions and all diseases are within the scope of Rad-X. Besides imaging, Rad-X also provides management of workflows for the reporting of examinations.

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It is being developed by Simplex Bilgi Teknolojileri A.Ş. Rad-X Development Team. It is planned to be used by radiology departments of hospitals and private radiology/imaging centers.

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This document is prepared to present the results of Rad-X's system sufficient test. It is intended for internal use. It will be shared with the customer side when necessary.

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3 Yetki ve Sorumluluk / Authority and Responsibility

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This procedure is the responsibility of the Technical Support Manager for its execution, monitoring, and continuation.

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The server where Rad-X Server Software is installed must be hosted in an infrastructure protected against power failure. It should be in a location that provides service in a way that is not affected by extreme environmental conditions (earthquake, flood, fire, etc.). Necessary security measures must also be taken to prevent computer viruses and malicious software. The customer must provide these items. "The installation of the Rad-X software is carried out on the servers by authorized personnel of Simplex Bilgi Teknolojileri A.Ş., either on-site or via remote desktop connection. During or after the installation, no CDs, DVDs, USB drives, or any other media containing the installation files are provided."

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

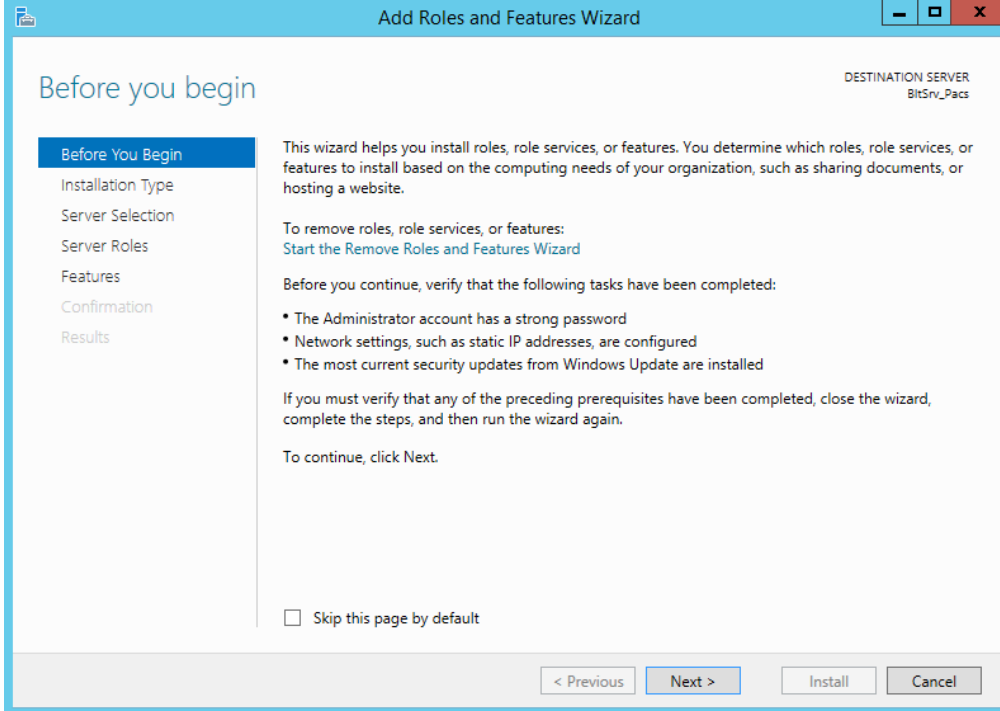
Control Panel → Programs and Features section opens. Following features are selected from 'Turn Windows features on or off' section. It is installed according to order in the images.

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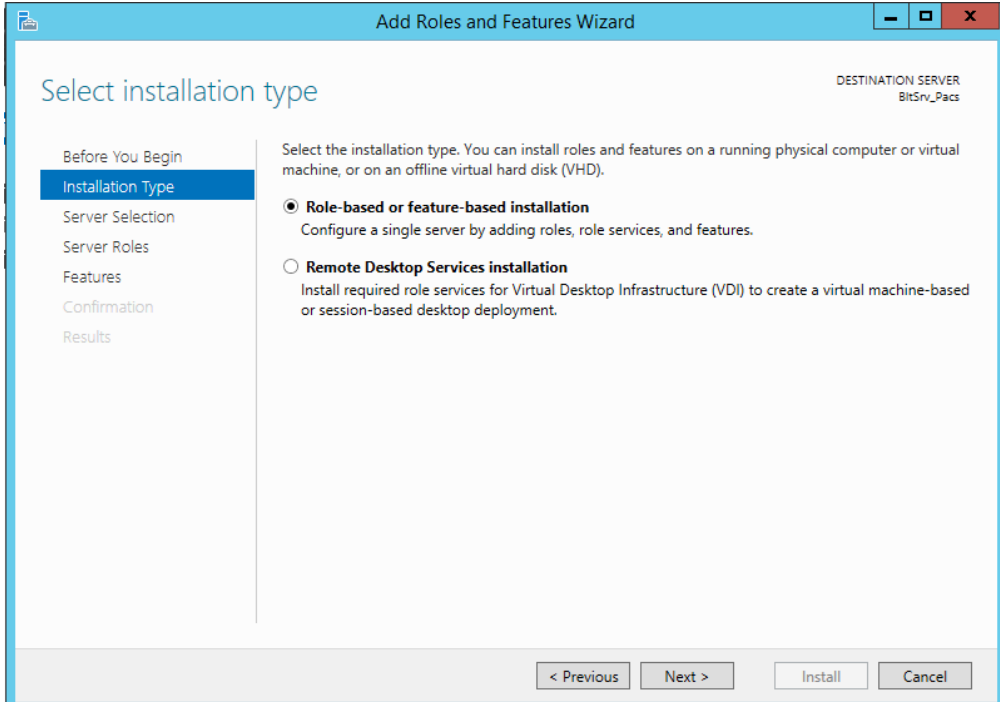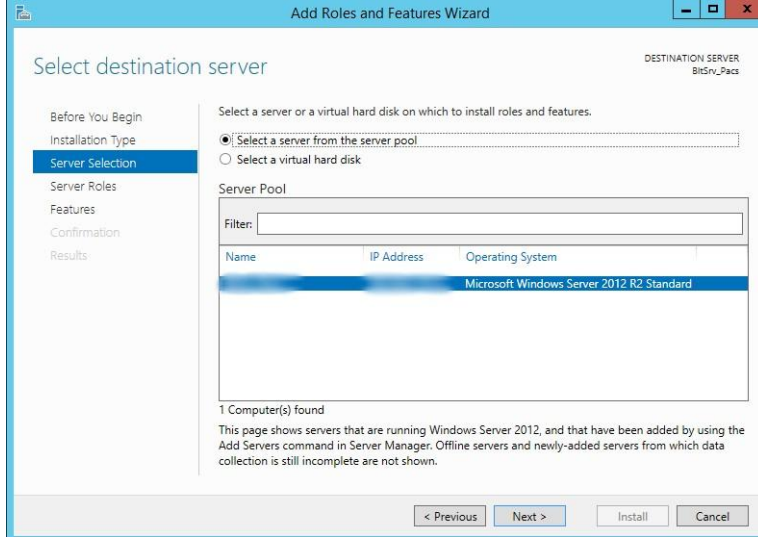
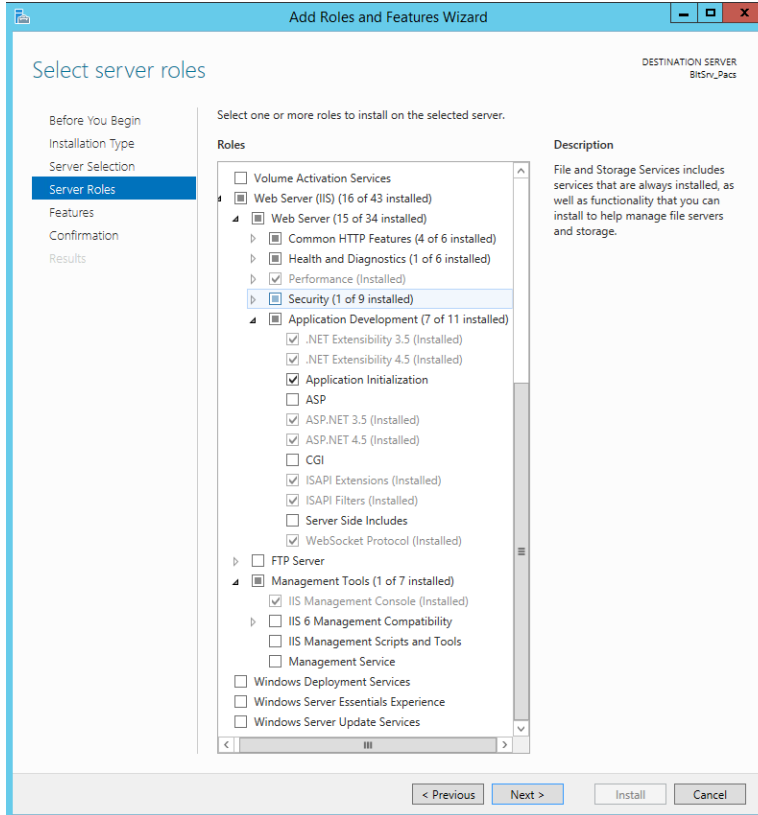
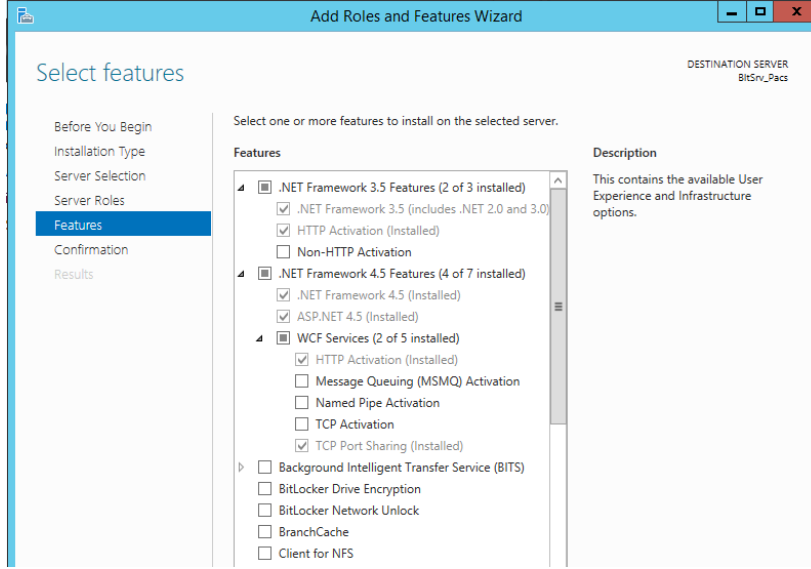
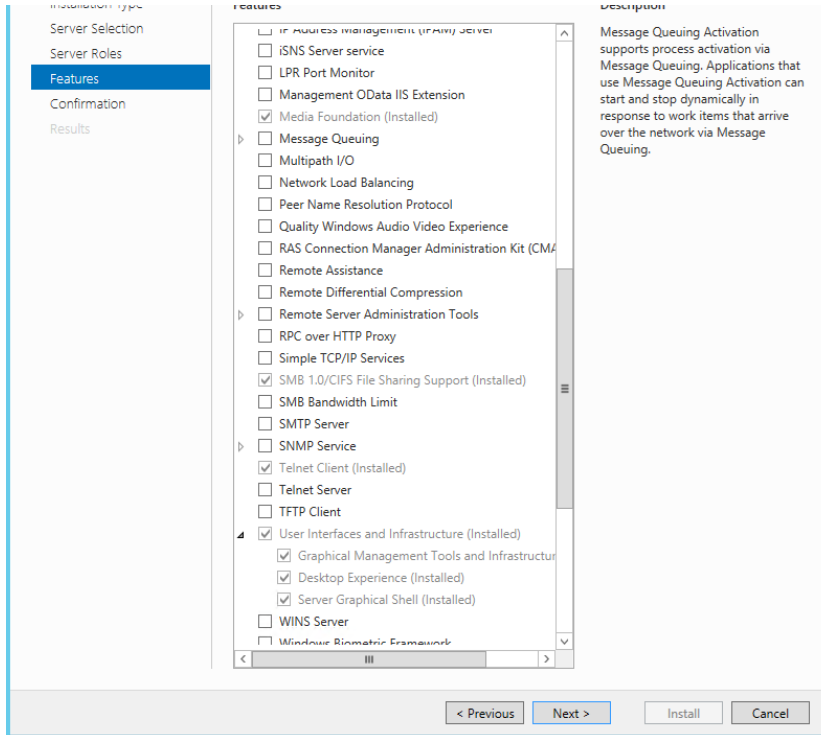
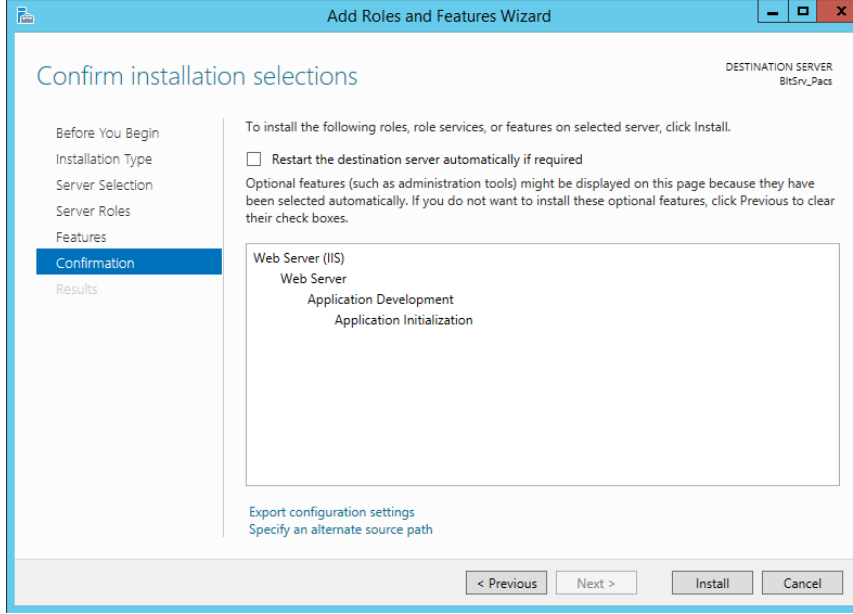
The features marked in Figure 5 and Figure 6 are marked. Then the Next button is pressed.

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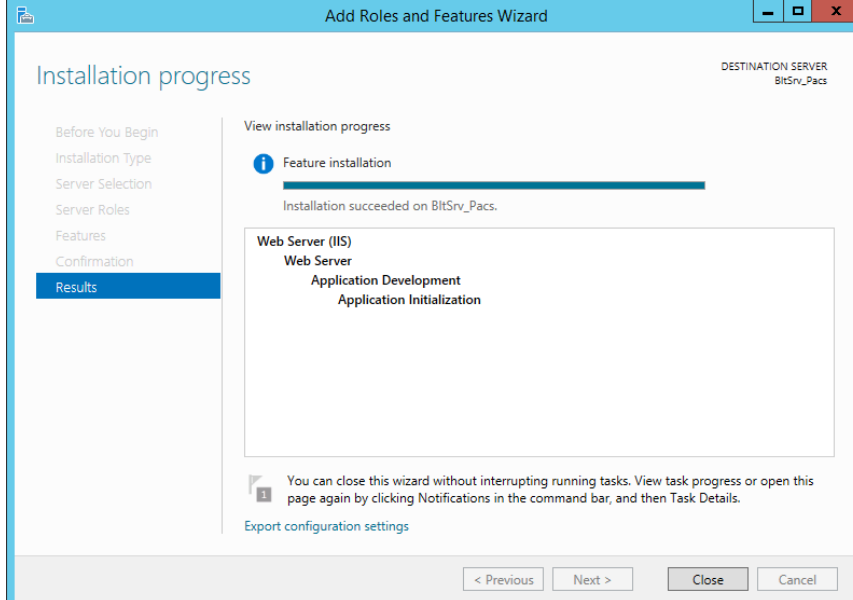When the confirmation screen (Figure 7) is reached, press the Install button.

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After the installation process is finished, press the Close button.

Doküman Kodu / Document Code
TF.01.IFU.01

Yayın Tarihi / Release Date
15.05.2024

Revizyon No/Tarih / Revision No / Date
01/15.01.2026

4.2.3 Microsoft Visual C++ 2015 Redistributable Update 3 kurulumu / Microsoft Visual C++ 2015 Redistributable Update 3 Installation

<http://msdn.microsoft.com> adresinden Microsoft Visual C++ 2015 Redistributable paketi indirilir ve kurulur. İşletim sistemi özelliğine uygun paket indirilmelidir. En son güncelleme sürümü indirilmelidir. It is downloaded and installed Microsoft Visual C++ 2015 Redistributable package from <http://msdn.microsoft.com>. The package suitable for the operating system feature must be downloaded. The latest update version must be downloaded.

4.2.4 Microsoft Visual C++ 2013 Redistributable kurulumu / Microsoft Visual C++ 2013 Redistributable Installation

<http://msdn.microsoft.com> adresinden Microsoft Visual C++ 2013 Redistributable paketi indirilir ve kurulur. İşletim sistemi özelliğine uygun paket indirilmelidir. En son güncelleme sürümü indirilmelidir. It is downloaded and installed Microsoft Visual C++ 2013 Redistributable package from <http://msdn.microsoft.com>. The package suitable for the operating system feature must be downloaded. The latest update version must be downloaded.

4.2.5 ASP.NET Core Runtime 2.0.9 Hosting Bundle kurulumu / ASP.NET Core Runtime 2.0.9 Hosting Bundle Installation

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It is downloaded and installed Hosting Bundle package from <https://download.microsoft.com/download/3/a/3/3a3bda26-560d-4d8e-922e-6f6bc4553a84/DotNetCore.2.0.9-WindowsHosting.exe>.

4.2.6 ASP.Net Core Runtime 3.1.11 Hosting Bundle kurulumu / ASP.Net Core Runtime 3.1.11 Hosting Bundle Installation

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It is downloaded and installed Hosting Bundle package from <https://download.visualstudio.microsoft.com/download/pr/d8b046b7-c812-4200-905d-d2e0242be9d5/53d5698d79013be0232152ae1b43c86b/dotnet-hosting-3.1.11-win.exe>

4.2.7 ASP.NET Core 4.8 Runtime kurulumu / ASP.NET Core 4.8 Runtime Installation

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It is downloaded and installed package from <https://download.visualstudio.microsoft.com/download/pr/014120d7-d689-4305-befd-3cb711108212/1f81f3962f75eff5d83a60abd3a3ec7b/ndp48-web.exe>

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* After adding IIS features and installing Runtime packages, the computer is restarted.

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The C:\PACS directory is created. Application files are moved under the PACS folder. The IIS user is authorized for this directory.

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*** The folder is clicked with the right mouse button. Properties text is selected. The user is found on the Security tab, the necessary permissions are given from the Edit option.

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Start -> Internet Information Services(IIS) Manager interface is opened. Click the right mouse button on the Sites text and click on the Add Website text in the opening window and the site name is written in the 'Site name' section in the opening window, as seen in Figure 9. C:\Pacs directory is selected in the 'Physical path' section. If there are any certificate and port details, they should be entered. Host name field is left blank and OK button is pressed.

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4.2.9 Auth Penceresi ve Servislerinin kurulumu / Installation of Auth Window and Services

C:\PACS\Auth dizinine girilir. / It is entered the C:\PACS\Auth directory.

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The file 'appsettings.json' is opened in notepad. In the opened notepad, database access information is written in the 'ConnectionStrings' section. In the 'AuthorityUri' section, the host address is written and '/auth' is added to the end. Ex: http://testadres.com.tr:80/auth. In the 'HttpSchemeValue' section, if the certificate was used in the IIS installation, "https" is written, if not, "http" is written.

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The file 'nlog.config' is opened in notepad. Check that the 'internalLogFile' section starts with C:\PACS\TEMP.

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In the 'name="allfile" ' section under the '<targets>' section, check that the 'fileName' and 'archiveFileName' sections start with C:\PACS\Temp.

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After completing the above operations, click on the auth text under our site in the IIS interface with the right mouse button. Click on Convert to Application. Then the connection address written in the 'AuthorityUri' section is accessed with an internet browser. After opening the auth page, it is seen that the auth database is created in the database.

4.2.10 API Servisleri ayarı / API Services setting

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It is entered the C:\PACS\api directory.

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'appsettings.json' file is opened in notepad. Database access information is written in 'DefaultConnection' section. The value written in 'AuthorityUri' section in appsettings.json file under auth directory is written in 'AuthURL' section. It is checked that the directories in the file start with C:\PACS\.

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'nlog.config' file is opened in notepad. It is checked that the directory address written in the 'internalLogFile' section starts with C:\PACS.

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In the target information named 'allfile' under the '<target>' section, it is checked that the directory address in the 'fileName' and 'archiveFileName' sections starts with C:\PACS.

4.2.11 HL7 Sunucu kurulumu / HL7 Server Installation

C:\PACS\hl7 dizinine girilir.

It is entered the C:\PACS\hl7 directory.

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'HL7MReceiver.exe.config' file is opened in notepad. The 'SPacsApiAddress' section is edited with our host name. Ex: 'http://testadres.com.tr:80/api/api/values/'.

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In the 'PortNumber' section, the port number information to be used is written.

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The 'IdentityServerBaseUri' section is adjusted according to the host address. Ex: 'http://testadres.com.tr:80/auth/connect/token'

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It is entered the C:\PACS\pacs server directory.

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'SPacs.Server.exe.config' file is opened in notepad.

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Database information is written in the 'DefaultConnectionString' section.

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It is checked that the directory names in the file start with C:\.

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In the 'WadoBaseUri' section, the address of our host and the port of our site are written. Ex: http:// testadres.com.tr:80

4.2.13 Service kurulumu / Service Installation

C:\PACS\service dizinine girilir.

It is entered the C:\PACS\service directory.

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'nlog.config' file is opened in notepad. It is checked that the directory addresses in the file start with "C:\".

4.2.13.2 SPacs.Service.exe.config dosyası ayarları / SPacs.Service.exe.config file settings

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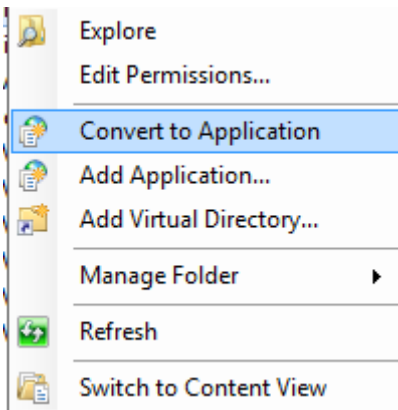
The file 'SPacs.Service.exe.config' opens in the notebook. Database connection information is written in the 'DefaultConnection' section.

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Start -> IIS interface, click on admin, api, help and viewer with the right mouse button and select Convert to Application from the pop-up window shown in Figure 10. It is observed that the application symbols change.

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4.2.15 Windows Servislerinin kurulması / Installation of Windows Services

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The command screen opens in the directory where the installation files are located. In the command window, the commands 'nssm install pacs.HL7Server' for the hl7 service, 'nssm install pacs.pacs_server' for the pacs service and 'nssm install pacs.service' for the service are written one by one and the appropriate programs are selected after the commands are written.

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Then, the services installed by typing Start -> window+r -> services.msc are started.

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The directory(s) for storing the images related to the application is created.

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The sample files located in the directory \\192.168.1.115\pacs\etc are transferred to the server using WinSCP. If an FTP server is not installed, it may be necessary to request its installation.

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The content of the default file and the configuration (domain name, port, certificate (pem, key, certificate file password)) must be discussed internally.

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Postgresql is installed according to <https://linuxconfig.org/ubuntu-20-04-postgresql-installation> link.

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Pgadmin is installed according to <https://www.pgadmin.org/download/pgadmin-4-apt> link.

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According to <https://docs.microsoft.com/en-us/dotnet/core/install/linux-ubuntu> bağlantısı 20.04, runtime Microsoft.AspNetCore.App 3.1.11 and Microsoft.NETCore.App 3.1.11 are installed.

Doküman Kodu / Document Code
TF.01.IFU.01

Yayın Tarihi / Release Date
15.05.2024

Revizyon No/Tarih / Revision No / Date
01/15.01.2026

`sudo apt-get install -y aspnetcore-runtime-3.1` ile kurulum yapılır.
The installation is performed with `sudo apt-get install -y aspnetcore-runtime-3.1`.

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```
.NET Core runtimes installed:
Microsoft.AspNetCore.App 3.1.11 [/usr/share/dotnet/shared/Microsoft.AspNetCore.App]
Microsoft.NETCore.App 3.1.11 [/usr/share/dotnet/shared/Microsoft.NETCore.App]
```

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4.3.7 Web projelerinin kurulumu (api, viewer, auth) / Installation of web project (api, viewer, auth)

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It is created /usr/share/nginx/html/dotnet directory. api, viewer, auth directories are copied under this directory.

The path written in the nlog.config files and the paths in the appsettings.json file are created, the files are copied.

Tüm uygulamaların log dizini Log directory of all applications	/home/simplex/apps/logs
api31	/home/simplex/apps/temp/sounds/
api31	/home/simplex/apps/downloadfiles/
api31	/home/simplex/apps/temp/dicoms/
api31	/home/simplex/apps/temp/pngs/
api31	/home/simplex/apps/temp/videos/
api31	/home/simplex/apps/appfiles/notificationfiles

Doküman Kodu / Document Code
TF.01.IFU.01

Yayın Tarihi / Release Date
15.05.2024

Revizyon No/Tarih / Revision No / Date
01/15.01.2026

auth	/home/simplex/apps/temp/clients.json
auth	/home/simplex/apps/temp/auth_signing.pfx

/etc/systemd/system altına api31.service, auth31.service, viewer.service dosyaları kopyalanır.
The files api31.service, auth31.service, viewer.service are copied under /etc/systemd/system.

```
cd /etc/systemd/system
```

```
sudo cp ~/files/systemd/system/* .
```

```
cd /usr/share/nginx/html/
```

```
sudo chown simplex dotnet -R
```

```
sudo chgrp simplex dotnet -R
```

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The following commands are executed.

sudo systemctl enable api31	sudo systemctl enable auth31	sudo systemctl enable viewer
sudo systemctl start api31	sudo systemctl start auth31	sudo systemctl start viewer
sudo systemctl status api31	sudo systemctl status auth31	sudo systemctl status viewer

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It is checked that the services are running.

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The directory /usr/share/simplexapps is created.

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pacsservice and service directories are copied.

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The directories in the appsettings.json file are created.

pacsservice	/home/simplex/apps/downloadfiles/
pacsservice	/home/simplex/apps/weasis-portable/
pacsservice	/home/simplex/apps/pathology/
pacsservice	/home/simplex/apps/autogenfolder/
service	/home/simplex/apps/temp/CadPath
service	/home/simplex/apps/appfiles/AI/model

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The files pacserver.service and service31.service are copied under /etc/systemd/system. The following commands are executed.

sudo systemctl enable pacserver	sudo systemctl enable service31
sudo systemctl start pacserver	sudo systemctl start service31
sudo systemctl status pacserver	sudo systemctl status service31

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